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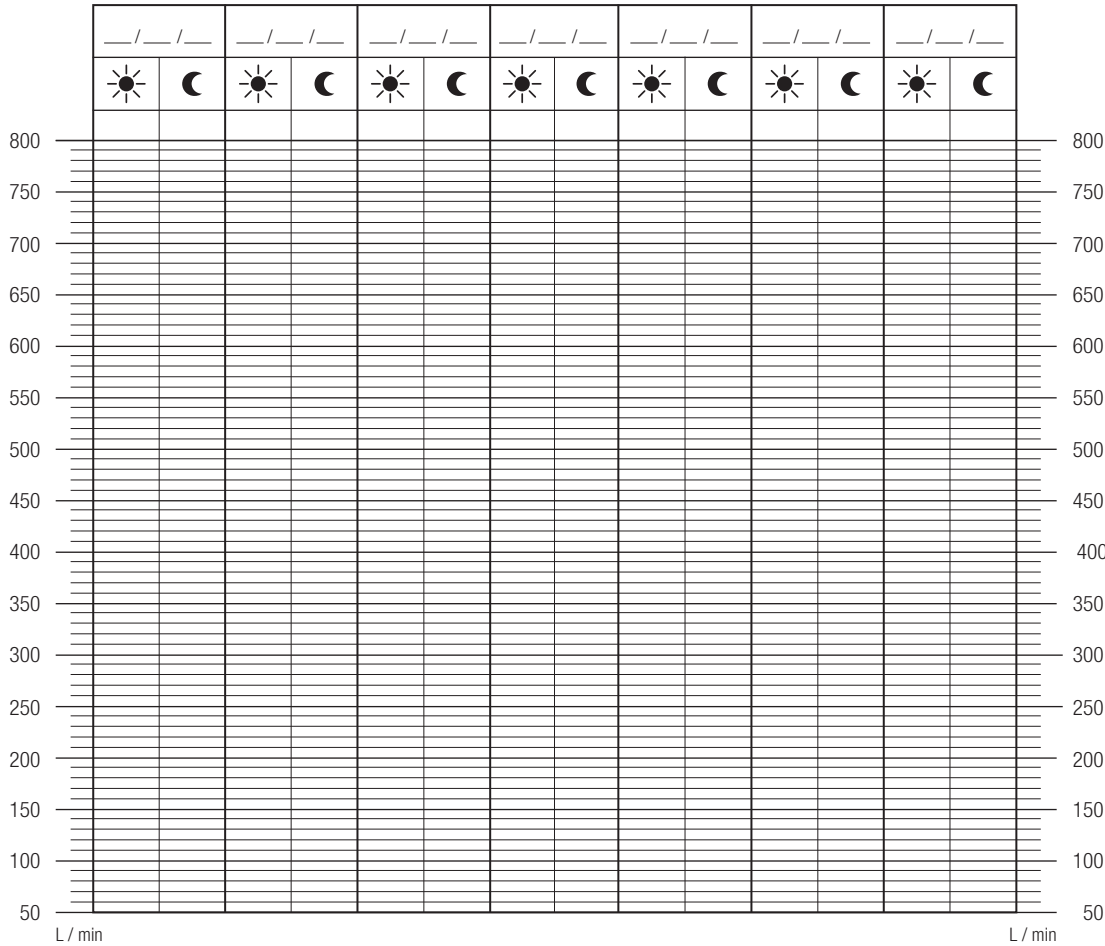
EN - Instructions for use

Your peak-flow meter

The Peak-Flow éolys meter is intended for use in the prevention and monitoring of asthma by measuring the expiratory peak flow of adults and children.

Restrictions of use

Do not blow as you would do into a blowpipe; do not obstruct the movement of the slider; do not obstruct the air outlet; do not tighten, loosen or lose the screws.

Instructions (see figures)

1. Place the slider in the blue zone.
2. Breathe in as deeply as you can, with your chin up.
3. Bite gently the mouthpiece.
4. Breathe out as HARD and QUICKLY as you can. Repeat it twice, resting between each blow.
5. After writing down the date, record your best result of your 3 blows, following the left-hand side axis. Measure your peak-flow capacity in the morning (☀) and another in the evening (☾). It is recommended to photocopy the grid before use.

IMPORTANT:

If you note a significant decrease in the results, contact your doctor.

Technical features

Accuracy: +/- 10 l/min or +/- 10 %.
Resistance to flow: < 0,35 kPa/l/s.
Units expressed in BTPS values.

Cleaning

The appliance must be regularly cleaned. Immerse it in a slightly soapy tepid water (not warm and never boiling). Rinse and let it dry.

Maintenance

The peak-flow meter does not require any particular maintenance apart from cleanliness. Keep it in a clean and dry place.

Dysfunction and troubleshooting

You are advised to check your measuring appliance on a regular basis to make sure that it works properly. Check that the cursor can move up and down normally and that all other parts remain fixed. If this is not the case you must send the appliance back to your supplier.

Disposal note

The device can be thrown away as a household waste.

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